

E-TEMP

HYDRAULIC TEMPORARY RESTORATIVE MATERIAL

NEW

- Temporary filling material for standard cavities and post-endodontic treatments.



Benefits

- Temporary Filling Material for Standard Cavities.
- Easily setting initiated by saliva.
- Excellent adaptability.
- Easy to remove.
- Adheres to dentine, expands slightly during the setting and hardens rapidly to ensure hermetical seal.
- Improved the mechanical properties for the quick temporary filling.

Directions


- Prepare and dry the cavity with a small cotton pellet.
- Transfer E-TEMP to the cavity using a suitable instrument and form the contour. In the case of occlusal Class I, the patient bites down on the surface, and then form the contour.
- After the patient close the mouth, E-TEMP will harden within 25 minutes. Recommend the patient to avoid the strong pressure or abrasion for at least 2 hours.

Package

- 30g ITEM #2004-1001




Setting Time

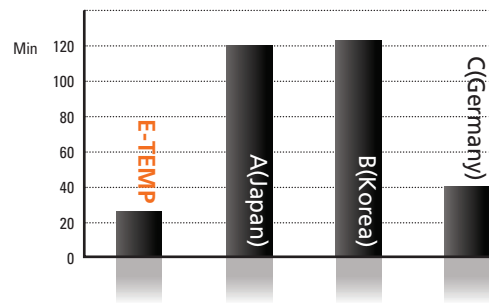

- This test is measured as ISO 3107 regulations after the same setting time in the laboratory(R&D DiaDent 2013)

Compressive Strength

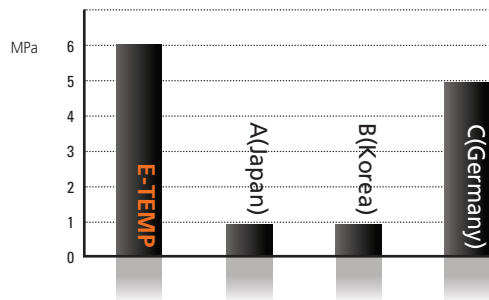

- This test is measured as the same condition at 37 degrees and 95% humidity in the laboratory(R&D DiaDent 2013)