

DIAPLUS™ G7 SINGLE COMPONENT SELF-ETCHING LIGHT-CURED BONDING AGENT

DiaPlus™ G7, the innovative 7th Generation bonding agent and offers superior and consistent bonding results on dentin and enamel with one bottle, one coat self-etching bonding agent system.




Benefits


- Has **Etch, Desensitize, Prime and Bond** all with One-Bottle, One-Coat bonding system and no separate etching
- **Fast 3 Steps** (less than 35 seconds) reduces the risk of contamination
- Offer a **tight and long-lasting bond** between light-cured composite and both enamel and dentin by a highly advanced formulation whether wet or dry on dentin and enamel
- **Strong and tight bond** to dentin and enamel is offered by 4-MET monomer for adhesive to dentin and Phosphoric Acid Ester Monomer for superior bonding strength to enamel
- Consistent and **a long term bond strength** with the unique Nano-Filler
- Provide a very low risk of post-operative sensitivity
- **Remarkable thin bonding layer** less than 10^{µm} offers a frosty surface for much easier application of the first composite layer

Refill Package

- **ITEM #2001-2702**
 - One bottle bonding agent (1 bottle x 5ml)

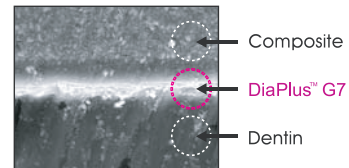


Thin Bonding Layer

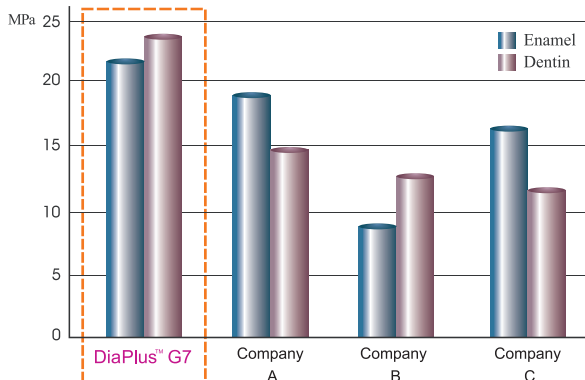

Superior Bond Strength

- One coat, simple and fast bonding application

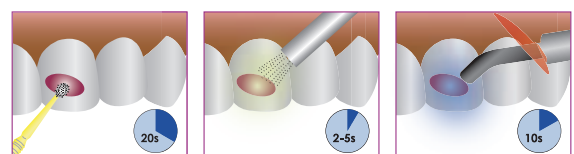

- 1** Apply Bonding Agent with scrubbing for 20 seconds.
- 2** Dry with high pressure air for 2-5 seconds.
- 3** Light cure for 10 seconds.

Less than 35 seconds

Technique Guide