

**DIA-ROOT™ BIO MTA****MINERAL TRIOXIDE AGGREGATE****NEW**

Root Repair Materials with excellent biocompatibility and offers distinct advantages for root canal repair procedures. DIA-ROOT™ BIO MTA is used to restore the damaged areas of root canal to normal function and appearance, quickly and efficiently.

**Indications**

- Repair of Root Resorption and Perforations
- Direct Pulp capping
- For Pediatric Pulpotomies
- Root-End Filling
- Apical plug during apexification

**Benefits**

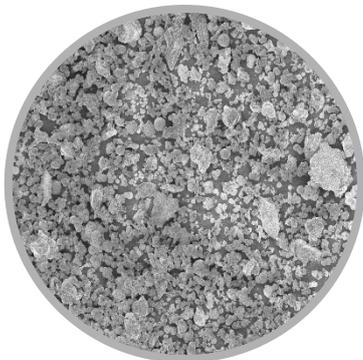
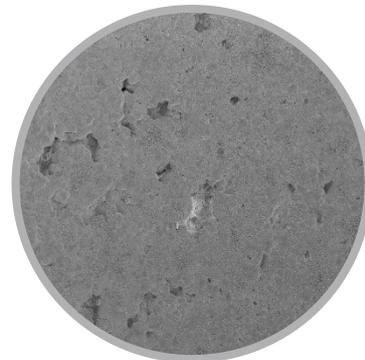
- Good Biocompatibility
- Outstanding Sealing Capabilities
- White powder is ideal for Anterior teeth
- Antibacterial Effect(pH 12)
- Easy Handling Properties
- Resists washout

**Directions**

- Open the cap of MTA powder
- Pour the powder into the mixing pad
- Put the distilled water onto the powder: Mixing Ratio 0.5g/0.2225g(Powder/Liquid)
- Wet the powder and mix it gently for few seconds
- Carry the mixture on the part for the treatment

**Packages**

- ITEM #1003-601
  - MTA Powder 0.5g
  - 1 Mixing Pad
  - 1 Spatula

**BEFORE****AFTER**

⟨Hydration⟩

SEM pictures of after Hydration DIA-ROOT™ BIO MTA show outstanding sealing ability